



**Model
ASL-325**



**Space-Saver™
Exterior Hitch Mount.**

OPTIONAL BRUNO POW'RPAC



- ◆ Easy to use!
- ◆ Convenient!
- ◆ Works in most minivans, SUVs, station wagons & sedans!

The complete, independent power supply for your automotive lift!

Pow'rPac easily installs in your vehicle – just plug into the vehicle's power outlet!

- ◆ The Bruno Model ASL-325 Space-Saver™ Vehicle Lift is an exceptionally versatile take apart lift designed to help store your scooter in vehicles with larger trunks or hatchbacks. The Space-Saver solves the problem of storing a scooter in smaller vehicles with limited rear space.
- ◆ The Space-Saver is designed to fit closer to the interior trunk wall, providing a 'tighter arc' when manually swinging the scooter in/out of the trunk. This allows scooters to be stored in spaces that won't work with more conventional lift designs, and the rotation of the lift arm makes it possible to raise/lower the scooter at the rear or side of the vehicle.
- ◆ The Space-Saver is fully-adjustable for height and length and raises scooters weighing up to 200 lbs. The lift tower base can be easily adjusted to fit nearly any larger car's trunk, ensuring a useful, reliable installation. The Space-Saver is lightweight, and assembly/disassembly can be completed in three simple steps, quickly and easily.
- ◆ The Space-Saver can be stored anywhere – in the trunk or inside the vehicle itself. And when an additional tower base installation kit is ordered, the Space-Saver can be used in more than one vehicle. Contact Bruno for details.
- ◆ The Bruno Exterior Hitch Mount is also available for the Space-Saver Lift. It's ideal for nearly any Coupe, Sedan, Hatchback, Wagon and Sport Utility Vehicle on the market, such as the Ford Focus ZX3 and ZX5, Honda CRV, Toyota RAV4... and more!
- ◆ Expert installation and complete, after sale service are provided by Bruno's worldwide network of dealers.
- ◆ Only Bruno has over 20 different interior/exterior scooter, wheelchair and power chair lifts for nearly any sedan, minivan, sport utility vehicle (SUV), full-size van, pickup truck and the new crossover vehicles.
- ◆ All Bruno lifts are eligible for reimbursement under the terms of Mobility Programs offered by most vehicle manufacturers.

Using the *Bruno Space-Saver™* is easy!

1



Once the scooter is positioned at the rear of your car, remove the seat and hook the *lift claw* to the T-bar docking device.

2



To lift the scooter, push the **UP** button until the wheels clear the trunk opening.

3



Once the scooter is at the proper height, manually rotate the scooter over the trunk opening.

4



With the scooter in position, push the **DOWN** button until the scooter rests on the trunk floor.

5



Disconnect the *lift claw* from the scooter's T-bar docking device and disassemble the lift. The heaviest piece weighs only 13 lbs. (6kg).

6



The three pieces of the Space-Saver Lift assembly can be stored with the scooter or inside your vehicle.

Bruno lifts are eligible for reimbursement under the terms of Mobility Programs offered by most vehicle manufacturers.



The Lift weight rating is the **MAXIMUM** load value. Factors inherent to specific applications including, but not limited to, vehicle structure and condition, operating environment, and customer ability may reduce the effective weight rating.



Represented By:

Bruno Independent Living Aids, Inc., 1780 Executive Drive
P.O. Box 84, Oconomowoc, WI 53066

Toll Free 1-800-882-8183 ♦ (262)567-4990
FAX (262) 953-5501 ♦ www.bruno.com